

Stress-Free Calf Tube Feeding



FLEXI TUBER – STARTER PACK


UNIQUE DESIGN & EASY TO USE

Designed by a Vet, the Flexi Tuber significantly reduces calf stress discomfort and injury is avoided. The flexible tube swallows easily – calf struggling is minimised making the process very easy to learn.

- Flexible Tube can be held in place with one hand
- Flexi Tuber offers a strap-on function for a more hands free option
- Large 4 Litre capacity bottle
- Transparent flexible tube makes it easy to clean & monitor flow
- Ergonomic bottle handle – perfect for large and small hands
- Reliable valve – fast flow even with thick colostrum
- Shown in trials to almost halve procedure feeding time.



Stopper & Calf size markers provide easy guide for tube placement & positioning

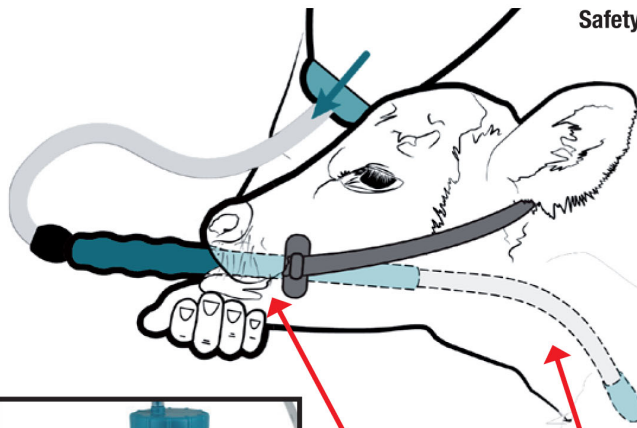

Safety tip



Mouthpiece crossbar sits comfortably in the corner of the calf's mouth. The strap is placed behind ears.



Clip on bottle keeps mouthpiece clean & away from surfaces.

Flexible tube is advanced through mouthpiece so the tube is easily positioned within the oesophagus.



Comes with Peach Teat cap for easy transition between teat and tuber.



1-866-527-6229
farmersfarmacy.com

HOW TO USE – FLEXI TUBER



2 Pass the flexible tube through the mouthpiece so tube enters the oesophagus.



3 Stopper and calf size guides for easy positioning.

When the tube has passed into the oesophagus it is safe to raise the bottle.



1

Mouthpiece crossbar sits comfortably in the corner of the calf's mouth. The strap is placed behind ears – fasten so mouthpiece sits snug in the mouth.

NUMBER ONE RULE:


Never let fluid flow unless the tube is safely in the oesophagus.

During Flow:

Hold the mouthpiece in place throughout feeding – hold with the calf so if the calf moves the mouthpiece moves with the calf. Do not let the tube pull out more than one inch during fluid flow. Never let the calf drop its head below the level of the stomach.

Removing the Tube:


Wait at least one second after all liquid has finished running through, then gently remove.

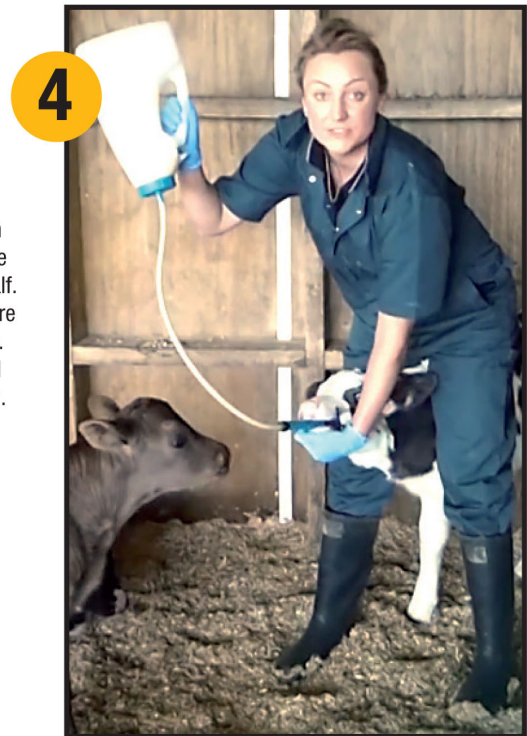

4

STOPPER POSITION & CALF SIZE MARKERS:

- Please consider the calf size markers a MINIMUM distance to pass the tube through.
 - The stopper should be positioned to the RIGHT HAND SIDE of the appropriate mark.
- Do not underestimate calf size as it is better to pass the tube farther than not far enough. If feeding a group of calves we recommend you set the stopper position to the largest calf in the mob.

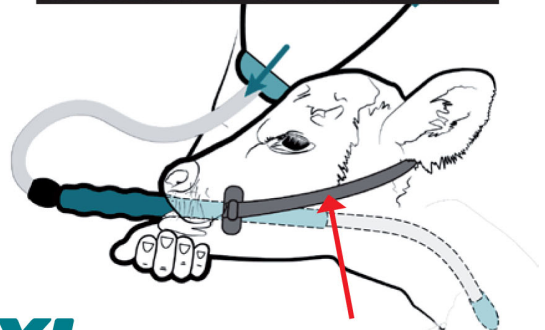

The strap is placed behind ears – fasten so mouthpiece sits snug in the mouth.

Please watch the demonstration videos and read our calf handling tips – www.antahi.com



FLEXI TUBER



1-866-527-6229
farmersfarmacy.com